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Via Tossicia 15
0013 Rome - Italie

STERILE A

DAILY EYE WASH AND IRRIGATING SOLUTION EXTRACT OF CHAMOMILE & EUPHRASIA

NAVI VISION™ is a line of based purified vegetal extracts products, in the form of ophthalmologic drops developed to fight the annoying symptoms caused by irritating situations for the fragile ocular structure, by long exposure to environments with air conditioned, by long hours of work in front of computer screens, smog, pollen, and by other non pathological factors.

CONTENTS:

Extract of Chamomile *Matricaria* 200 mg & extract *Euphrasia officinalis* 200 mg in a buffered isotonic solution with pH 7.3 q.b. in 100 ml.

INDICATIONS:

The **NAVI VISION™ Eye wash** helps to relieve: Irritation, discomfort, stinging, itching by the Removal of foreign bodies in the eye. Air pollution such as smog, pollen, atmospheric and calms irritated eyes due to the excessive use of contact lenses.

FOR EMERGENCY USE:

Flush abundantly the affected eye with the solution. Do not touch the eye with the tip of the bottle

FOR DAILY USE AS AN EYE WASH:

Fill in the cup with the solution and bath the affected ocular surface.

It is very important keep the cup clean at all times and when re-using it, rinse well with the **NAVI VISION™** solution.

WARNINGS:

- Do not use if you have open wounds in or near the eyes
- If the solution changes color or becomes cloudy
- Can be used with the contact lenses on but make sure that you do not loose them during the irrigation process.
- Do not touch the tip container to any surface
- Close the bottle after use
- Wash and disinfect the eye cup with the solution before and after use
- Consult your eye specialist if you have eye pain, continued irritation and redness or if the condition worsens and does not get better
- Keep out of reach of children

STORAGE:

Store solution between 5° et 35°C.

DIRECTIONS OF USE:

The **NAVI VISION™ Eye wash** can be used with the eye cup or without which we recommend to avoid eye contamination. Maintain the eye cup always cleaned by rinsing with the **NAVI VISION™** solution before and after using it.

WITHOUT CUP

WITH CUP

240 ml - Sterile

Navi Pharma

Lot 10, Les Horizons Bleus
391 Chemin des Groules
06600 Antibes - France

Manufacturer:

Omisan Farmaceutici
Via Tossicia 15
0013 Rome - Italy

0123

STERILE | A