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 **Omisani Farmaceutici**
Via Tossicia 15
00131 Rome - Italy

STERILE A

LUBRICATING OPHTHALMIC SOLUTION WITH 0.3 % SODIUM HYALURONATE AND ESSENTIAL ELECTROLYTES

COMPOSITION:

NAVIAL® 10 ml bottle

Hyaluronic acid sodium salt 0,3 %, PHMB, sodium chloride, potassium chloride, calcium chloride dihydrate, magnesium chloride hexahydrate, borate buffer, purified water.

DESCRIPTION:

NAVIAL® is a lubricating ophthalmic solution, containing sodium hyaluronate 0.3%. The sodium hyaluronate contained in the product is obtained by fermentation and is not of animal origin.

The presence of electrolytes (chlorides, sodium, potassium, calcium and magnesium), essential for the cellular biochemical processes, and the slight hypotonicity of the solution, help to keep the ocular surface in physiological conditions.

NAVIAL® has a natural formulated preserving system, developed in the NAVI PHARMA laboratories that does not irritate the ocular mucosa and keep healthy the epithelial cells.

NAVIAL® maintains and restores the physiological conditions of the ocular surface, protects its cells with a persisting action and prevents the onset of metabolic disorders in the corneal epithelium.

INDICATIONS:

NAVIAL® brings rapid and lasting relief in cases of sensation of dry, irritated and tired eyes which are not caused by pathological conditions but induced by external factors such as prolonged exposure to video screens and monitors, air conditioning, dust, pollution, smoke, etc.

NAVIAL® is indicated for wearers of all types of contact lenses and non-wearers.

WARNINGS:

- To prevent possible contamination, do not let the tip of the dropper come into contact with any surface.
- Do not use in case of specific hypersensitivity towards any of the constituents.
- If the ocular irritation persists and you experience pain, redness or change in vision following the instillation, then consult your specialist.
- Do not swallow the solution.
- Keep out of reach of children.
- Store the bottle at a temperature below 35°C.
- Do not freeze.
- Do not use the solution after the expiry date.
- After the bottle has been opened, the solution must be used within 90 days. Any residual product must be disposed of after this date.

In case of persisting problems, consult your specialist.

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