VISIONLUXTM

SOLUTION

61

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GB SPECIAL FEATURE:

if used daily it prevents the deposit of proteins.

Visionlux is a multi purpose solution recommended for good maintenance and disinfection of your soft contact lenses (including frequent replacement and disposable lenses). The following 5 steps for daily maintenance and disinfection of your soft contact lenses are very important to ensure healthy eyes and a good optical vision. Another important benefit of **Visionlux** multi purpose solution is that when used daily and in accordance with the instructions it will prevent the deposits of proteins which can damage your lenses and may lead to eye infections and reduction of the lens life and wearing time.

THESE 5 STEPS ARE DESCRIBED BELOW:

Before starting the handling of your lenses always make sure that you wash, rinse and dry your hands.

1/CLEANING:

Remove the right lens. Place it in the palm of your hand. Add sufficient **Visionlux** multi purpose solution to bathe the lens. Gently rub both surfaces of the lens with the top of your forefinger for 15-20 seconds. Make sure that you do not scratch the lens with your finger-nails.

2/RINSING BEFORE DISINFECTING:

Rinse the lens thoroughly with the **Visionlux** multi purpose solution to remove the debris. Place the lens in the "R" (right) compartment. Repeat these steps for the left lens and place it in the "L" (left) compartment.

3/STORING AND DISINFECTING:

- Fill the lens case with **Visionlux** multi purpose solution.

- Screw the top until closed. Shake gently to ensure all the internal surfaces and lenses are wetted with the **Visionlux** multi purpose solution.

- Leave to disinfect for a minimum of 4 hours.
- The lenses will then be ready to wear.
- If your lenses have been stored for more than 7 days then you must re-disinfect them before using them.
- Each week rinse the lens case with the **Visionlux** multi purpose solution, open the lens case and let it dry. To minimize the opportunity for bacterial contamination of your lenses use a new lens case every month. Always discard the old lens case.

4/RINSING AFTER DISINFECTING:

Before wearing the lenses it is important to rinse your lenses with the **Visionlux** multi purpose solution to remove any residual debris.

5/LUBRICATING:

Use a few drops of the **Visionlux** multi purpose solution on each lens to help the fitting of the lens.

SPECIAL PRECAUTIONS:

- Do not mix with other solutions as it may cause discomfort and harm to your eyes and damage your lenses.
- In the event of a persistent irritation, remove the contact lenses from your eyes and consult your contact lens specialist.
- Consult your contact lens specialist before using any medication for your eyes together with your contact lenses.
- Do not use the solution if you are allergic to any of the ingredients.
- Keep all contact lens care products out of the reach of children.
- Do not allow the nozzle tip to touch any surface to avoid contamination of the **Visionlux** solution.
- Replace the bottle cap after use.
- Store below 30°C in a dry place.
- Discard the solution 90 days after opening. Do not use it after the expiring date.
- Do not rinse your lenses or contact lens case with tap water. Do not use maintenance products for your contact lenses that are not approved.
- Do not swallow.

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