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Italdevice srl
Via Laurentina Km 26,700
00040 Pomezia - Italie

CE
0373

ANTIOXIDANT AND LUBRICANT OPHTHALMIC SOLUTION WITH SODIUM HYALURONATE, VITAMIN A AND VITAMIN E

PROTECTS, SOOTHES AND RESTORES OCULAR SURFACE HEALTH

For topical ophthalmic use only
15 ml eye drops bottle

DESCRIPTION:

NAVITAE™ eye drops contains lubricant substances of polysaccharide origin such as hyaluronic acid and carboxymethyl beta-glucan, and a protective lubricant lipophilic system composed of pegylated Vitamin A and Vitamin E.

The topical ocular use of Vitamin A improves contact lenses wearing by reducing the mechanical damage that they can cause on epithelial cells of the cornea and conjunctiva.

Vitamin A stabilizes the fatty phase of lachrymal film by increasing its capacity to maintain natural hydration, therefore reducing the symptoms of ocular dryness. This activity is also exerted by the polysaccharides present in the product.

Vitamin E is an antioxidant that helps to reduce the concentration of free radicals in the lachrymal fluid, and therefore enhances the protection of corneal and conjunctival endothelial cells, decreasing the risk of Inflammation and promoting healing processes.

INDICATIONS:

NAVITAE™ eye drops is a lubricant ophthalmic solution with an antioxidant and moisturizing effect used for relief of dry eyes and ocular discomfort caused by:

- Ocular irritation caused by pathologies such as keratitis, adverse effects following corneal surgery, traumatic ulcers, corneal inflammation.
- Oxidative stress due to light exposure and/or increase of free radicals on the ocular surface.
- Contact lens wearing.

STERILE A

DIRECTIONS FOR USE:

1 or 2 drops in each eye, 3 to 6 times a day, or according to medical prescription.

COMPOSITION:

Sodium hyaluronate, carboxymethyl beta-glucan, vitamin A palmitate, vitamin E, TPGS (D-alpha tocopheryl polyethylene glycol 1000 succinate), sodium edelate, sodium N-hydroxymethylglycinate, buffered isotonic solution pH7,0.

SPECIAL PRECAUTIONS:

- This product is sterile until opening of the bottle.
- Close the container well after use.
- Do not use the product 30 days after opening of the bottle.
- Do not use the product after the expiry date indicated on the bottle.
- Keep out of reach of children.
- Do not swallow the solution.
- Do not use the product in case of allergy to one of the ingredients; in case of adverse reaction, stop using the product and consult your eye specialist.
- Do not let the tip of the dropper come into contact with your eyes.

STORING CONDITIONS:

- Store at room temperature and below of 35°C.
- Should not be exposed to a direct source of light .

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